



Seal Test sealer for testing the quality of seals, 30-pack



Available

SKU: DEA4.18

Categories: Sterilization test

Description

The Seal Test Sealer, thermal welding tests allow you to check the quality of the seals performed during the packaging of the instruments.

It's of the utmost importance that the packaging maintains the sterility of the material, until the established deadline in accordance with UNI EN ISO 11607-2 AND UNI EN 868-5 standards.

For this reason it is necessary to provide routine checks in sealing machines.

The Seal Test Sealer ascertains the quality of the thermal welding by highlighting the temperature and pressure parameters exerted on the bar on a specific area printed on the test.

The correct fusion of the inner layer of the film and the correct pressure on the paper of the bag are indicated by a uniform coloring in a specific area of the test.

Insufficient temperature, worn welding bar or insufficient pressure are the causes of a disabling result highlighted by the test.

- Ideal for daily testing of the sealing integrity of sterilization bags;
- 30 test sheets pack;
- In compliance with EN ISO 11606-2 standard;
- Checking consists of monitoring bars temperature and pressure. Test holds in evidence the uniformity of the weld as well as the perfect tightness of the flaps.